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EN **Orotol® plus Disinfection of Suction Systems**

Orotol® plus is a highly effective concentrate for the simultaneous disinfection, deodorisation, cleaning and care of all dental suction systems as well as spittoon bowls, and is suitable for all amalgam separators. The selected components of disinfectants and cleaning agents provide intensive, non-foaming care, which is free still gentle on materials and environmentally friendly. Orotol® plus is free of aldehydes and free of phenols, as well as free of active chlorine.

Composition: 100 g Orotol® plus contain 4.4 g dimethyldioctylammonium chloride, 0.6 g benzyl-dimethyl-dodecyl-ammoniumchloride, 5 - 15 g alkaline cleaning agents, < 5 g complexing agents, special anti-foaming agents, hexyl cinnamal, fragrances, adjuvants.

Spectrum of activity: Bactericidal, levurocidal, with limited virucidal activity (enveloped viruses, including HBV, HCV, HIV and coronavirus), non-enveloped adenoviruses and noroviruses. Orotol® plus is VAH-listed, Tested under high burden in accordance with EN 13727, EN 14561, EN 13624, EN 14562, EN 14476, EN 17111, EN 16615 as well as DVV/RKI [Association for the Control of Viral Diseases/Robert Koch Institute] guidelines.

Use: Orotol® plus is used at a 2% concentration. For example, use the measuring cup in the cap to dose 20 ml of Orotol® plus (marking cor-

responds to 20 ml) and fill and aspirate with cold water to 1 litre, e.g. using the OroCup. In accordance with VAH, the exposure time in 2% concentration is 5 minutes (bactericidal / levurocidal). Inactivation of viruses takes place at 2% - 30 min to 60 min (see table).

We recommend cleaning and disinfecting suction systems once or twice a day, according to work-load. Once-daily cleaning and disinfecting should be done at the end of the day. If necessary, clean and disinfect spittoon bowl with Orotol® plus, too. Follow the device manufacturer's instructions and country-specific requirements. For further information please also see product information on the Internet.

Caution: After surgical procedures, aspirate a glass of cold water inter-mittently. No foaming agents must enter the suction system or the spittoon bowl. Do not mix Orotol® plus with other products. Changes in the colour and odour of Orotol® plus during storage do not impair the efficacy. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

  see imprint.

Danger. Peut être corrosif pour les métaux. Causes severe skin burns and eye damage. Harmful to aquatic life with lasting effects. Wear protective gloves and eye/face protection. If skin irritation occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Rinse skin with water/shower. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

UK-REP: DÜRR DENTAL (Products) UK Ltd., 14 Linnell Way - Telford Way Industrial Estate, Kettering, Northants NN16 8PS, United Kingdom, info@duerruk.com

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