# FD 312 wet wipes

DE	Flächendesinfektion	DA	Overfladedesinfektion
EN	Surface Disinfection	CS	Dezinfekce ploch
FR	Désinfection des surfaces	SK	Dezinfekcia ploch
PT	Desinfecção de superfícies	EE	Pindade desinfitseerija
SV	Yt-desinfektion	LV	Virsmu dezinfekcijas lōrdzeklis
FI	Pintojen desinfiointiaine	LT	Paviršių dezinfekcija
NO	Overflatedesinfeksjon	RO	Dezinfectia suprafetelor

 rochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim, DE  
a DÜRR DENTAL company  
Tel. +49 7142 705-0  
info@duerrdental.com  
www.duerrdental.com  
Made in Germany

 **DÜRR  
DENTAL**

# DE FD 312 wet wipes

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*Recommended application*

*for surface disinfection according to VAH*

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<i>Virucidal activity against enveloped viruses</i> <i>(enveloped viruses incl. HBV, HCV, HIV</i> <i>as well as Coronaviruses)</i>	DVV/RKI-Leitlinie / DVV/RKI Guideline	60 min.
--	--	---------

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## EN **FD 312 wet wipes** Surface Disinfection

FD 312 wet wipes are ready-for-use cloths for the cleaning and disinfecting of floors, practice inventory etc. in accordance with EU Biocidal Products Regulation (EU-BPR).

**Composition:** FD 312 wet wipes contain FD 312. 100 g solution of active substance contain 65 mg alkyl-benzyl-dimethyl-ammonium chloride, non-ionic surfactants, alkaline cleaning components, complexing agents and additives in aqueous solution.

**Effective spectrum:** Bactericidal, yeasticidal, virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV and coronaviruses). FD 312 is **VAH-listed**. Tested under high load in accordance with EN 13727, EN 13624, EN 16615.

**Application: PT 2.** Remove FD 312 wet wipes individually. Wipe the surfaces and objects to be disinfected and allow the product to take effect. Disinfect visibly contaminated surfaces with FD 312 wet wipes and remove contamination. Disinfect cleaned surfaces again. Make sure that wetting is complete throughout the entire action time. Close the flow pack immediately after removing a wipe. In accordance with VAH, the action time is 15 minutes (bactericidal / yeasticidal), inactivation of enveloped viruses takes place within 60 minutes (see table). 1 wipe is sufficient for approx. 6-8 m<sup>2</sup>. For further information please also see product information on the internet.

**Close the storage pack properly after removing each cloth.**

**Note:** Dispose of large quantities in accordance with official regulations, if necessary dispose of as problem waste. Packaging should be recycled. Dispose of used cloths in the household waste – do not throw them down the toilet. In the case of surfaces with unknown material composition, observe the manufacturer's specification and/or check the resistances at an inconspicuous place. Changes in the colour and odour of FD 312 wet wipes during storage do not impair the efficacy. Use exclusively by qualified personnel.

  see imprint.

Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF ON SKIN: Wash with plenty of water.

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orochemie GmbH + Co. KG